

DR. BOEHM GmbH

Tizianstr. 33
D-90453 Nuernberg


Telefon: +49 (0)9 11 - 6 32 78 25

Fax: +49 (0)9 11 - 63 53 25

E-Mail: info@Dr-Boehm.de

Internet: www.Dr-Boehm.de

side 1

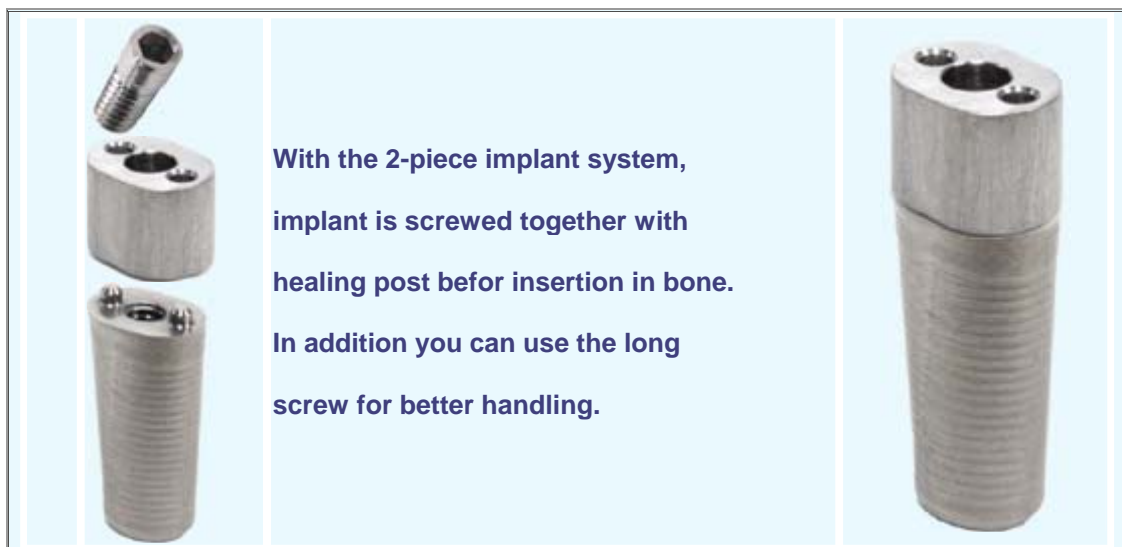

	<p>After healing of the gingiva the impression post is fixed with the long screw together with healing post that remains on implant.</p>	
	<p>Use impression spoon with hole. After age of impression remove the long screw. In laboratory a steel implant is fixed with according healing post as you can see on the left and fixed together with a long screw and impression post. After pouring with plaster (CaSO_4) and age the long screw is removed.</p>	



After that in laboratory the healing post removed and abutment is fixed with laboratory implant. Use short screw.



After finishing of crown in laboratory the dentist removes the healing post from implant in mouth and the abutment is fixed with short screw. Now the crown can be cementated.

